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Dear Sir or Madam, thank you for purchasing our product!

Before using the product, please read the following instructions to ensure proper use of the product.

Please keep this manual for future reference and follow its recommendations, as failure to follow its instructions may pose a threat to life or health.

APPLICATION AND DESCRIPTION

The ear cleaner with a soft, spiral tip is a practical tool designed to safely and effectively remove excess earwax. The flexible design allows for easy insertion of the tip into the ear and gentle rotation for thorough cleaning.

The device is intended for home use only and is not intended for medical or commercial use and is not a substitute for medical therapy. The device is intended for indoor use in dry rooms. It is not intended for medical or therapeutic use or for commercial use.

TIP

- The device must be checked for completeness of delivery and any visible damage.
- In the event of an incomplete delivery or if you notice damage due to faulty packaging or transport, please contact the service hotline.

TECHNICAL DATA

- Material: plastic, silicone
- Perfect for traveling
- Painlessly removes water and dissolved secretions
- It has silicone tips
- Safe for the ear
- Cleans very effectively
- 16 silicone tips included

SAFETY INSTRUCTIONS

- Before installing and using the product, please read this instruction manual carefully.
- This product may be used by children aged 8 years and over and by persons with reduced physical, sensory or mental capabilities or lack of experience or knowledge, provided they are supervised or have been instructed on the safe use of the product and understand the resulting risks.
- Children must not play with the product.
- Cleaning and maintenance may not be performed by children without supervision.
- Make sure that packaging materials are not left unattended. Children may start playing with them, which is dangerous.
- Keep the foil packaging of the product away from children (risk of suffocation)
- Avoid sources of ignition: avoid contact with flames or heat-generating devices (may cause fire)
- Protect the product from extreme temperatures, direct sunlight, high humidity, flammable gases, solvents.
- For cleaning, use a damp cloth or a mild detergent.
- Do not use a damaged device.
- If safe operation is no longer possible, discontinue use and secure the product against further use. Safe operation is no longer possible if the product: - has been damaged, - does not function properly, - has been stored for an extended period under unfavourable conditions or - has been excessively stressed during transport.
- It is prohibited to use the product if any part is damaged.
- Clean your ears gently, without excessive pressure.
- Do not insert the tip deeper than recommended (only into the visible part of the ear canal).
- Replace the tip after each use to maintain hygiene.
- Do not use the device if you suffer from ear inflammation, pain or infection.
- The product does not replace a medical consultation – in case of ear problems, consult a doctor.
- The cleaner is intended for personal use only.
- Always use the product as intended.

TIPS AND INFORMATION ON THE MANAGEMENT OF USED PACKAGING

The packaging is made of environmentally friendly materials that can be disposed of at your local recycling center.

Used packaging material should be delivered to a waste disposal point designated by local authorities. Information on the possibilities of recycling the used product is provided by the municipal or city office.

The product meets European and national requirements on the safety of devices and products.
We reserve the right to make changes to text, design and product data without notice.

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