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Dear Sir or Madam, thank you for purchasing our product!

Before using the product, please read the following instructions to ensure proper use of the product.

Please keep this manual for future reference and follow its recommendations, as failure to follow its instructions may pose a threat to life or health.

DESCRIPTION

The orthopedic pillow is an innovative solution designed to provide maximum support for the neck, nape and shoulders. Thanks to its ergonomic shape and properly matched filling, it helps maintain the natural curvature of the spine, which promotes healthy and comfortable rest. It is perfect for both sleep and short moments of relaxation at home, in the office or on the go.

APPLICATION

- Reduction of tension and pain
- Healthy sleep
- Relaxation and regeneration
- Rehabilitation support
- Preventing posture problems

TIP

- The device must be checked for completeness of delivery and any visible damage.
- In the event of an incomplete delivery or if you notice damage due to faulty packaging or transport, please contact the service hotline.

TECHNICAL DATA

- Removable cover
- Length: 40cm
- Height: 11cm
- Height at the indentation: 9cm
- Width: 22cm

SAFETY INSTRUCTIONS

- An orthopedic pillow is designed to support your neck and nape while you sleep or relax. Do not use it in a way that may cause improper support for your body.
- If the pillow causes discomfort, pain or muscle tension, discontinue use and consult your doctor or physical therapist.
- People with serious back problems, neck injuries or chronic pain should consult a specialist before using the pillow.
- Avoid using the pillow on skin areas with open wounds, inflammation or irritation.
- Make sure the pillow is positioned properly when in use, supporting the natural curve of your neck and nape.
- The pillow is not a toy. Children should only use it under adult supervision to avoid misuse.
- The pillow should not be used while driving or operating machinery so as not to restrict head movement.
- The cushion works best on flat, stable surfaces. Using it on uneven surfaces may reduce its effectiveness and comfort.
- Before each use, make sure the cushion is not damaged (e.g. tears, filling deformation). In case of damage, discontinue use.
- The pillow is not resistant to high temperatures and may be damaged by contact with fire or radiator.
- Do not squeeze or fold the pillow by force
- Too much pressure or folding may damage the filling or cushion material.
- If you notice an allergic reaction to the material or filling of the pillow, stop using it immediately and consult your doctor.
- Initially, use the pillow for a shorter period of time to allow your neck and nape to become accustomed to the new type of support.
- The pillow is designed to support the neck and nape of the neck only and should not be used to support other parts of the body.

STORAGE, CLEANING

- The pillow should be stored in a room with low humidity to avoid the growth of mold and bacteria.
- Avoid direct sunlight
- Do not leave the pillow in the sun for long periods of time as UV rays may damage the material or filling of the pillow.
- If you do not use the pillow for a long time, place it in a cover to protect it from dust and dirt.
- Avoid storing your pillow near radiators, stoves or other sources of high heat that may damage the foam or material.
- Store the pillow in a position that prevents it from deforming. Do not place heavy objects on the pillow.
- Air the pillow periodically to refresh the filling and remove moisture.
- Wipe the surface of the cushion with a damp cloth and mild detergent. Avoid soaking in water to avoid damaging the filling.
- Use mild detergents for washing.
- The pillow filling (e.g. memory foam) is not intended for washing. Contact with water may permanently damage the structure of the material.
- After cleaning, make sure the cover or surface of the cushion is completely dry before using it again. Dry in a ventilated place, avoiding direct contact with a heater or sun.
- Do not use bleach, solvents or strong detergents that may damage the material or destroy the properties of the filling.
- Regularly check the pillow for dirt, mold, or unpleasant odors. If necessary, freshen up the pillow with a disinfectant fabric spray (safe for the materials used).
- If the pillow has a removable cover, wash it according to the instructions on the label. It is usually recommended to wash it on a gentle cycle at a maximum temperature of 30°C. Use mild detergents.

TIPS AND INFORMATION ON THE MANAGEMENT OF USED PACKAGING

The packaging is made of environmentally friendly materials that can be disposed of at your local recycling center.

Used packaging material should be delivered to a waste disposal point designated by local authorities. Information on the possibilities of recycling the used product is provided by the municipal or city office.

The product meets European and national requirements on the safety of devices and products.

We reserve the right to make changes to text, design and product data without notice.

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